



## ***Manual: Assembly of Bottle Holder on Walker 12er size S***




**1** Place of assembly of the bottle holder (R or L).



**2** Wrap the strap around the frame tube and pull it through the holder.



**3** Pull the strap tight and turn the nut clockwise to tighten the strap.

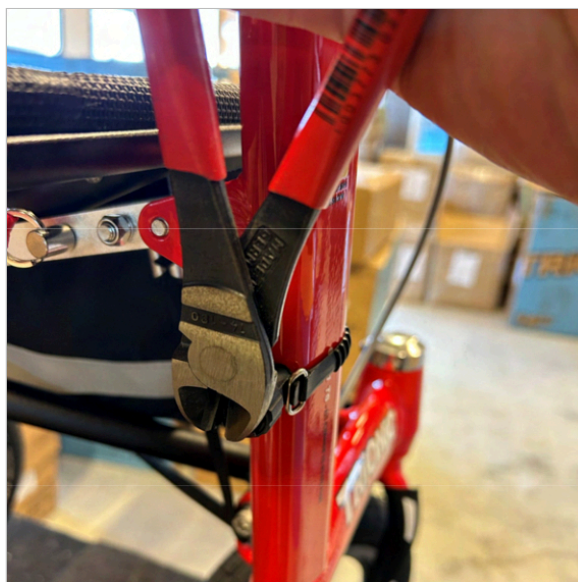

**4** Cut the excessive strap off.



5 Mount the second holder in the same way.



6 The two holders correctly assembled on the frame tube.



7 Attach the bottle holder with the two screws. The "opening" of the bottle holder should face outwards.



8 The bottle holder is now attached to the frame and you enter the bottle from the side and remove it outwards.