



## Comfort Folding Stool - User Manual

3-legged Comfort Folding Stool - Seat height 55cm/22" - Compact, lightweight and easy to bring along when heading outdoors - Including carry bag - Max load 225 kg

Skannaa QR-koodi älypuhelimellasi tai laitteellasi nähdäksesi video-ohjeet.



- The Comfort Folding Stool follows you wherever you go.
- It is compact enough to carry in your bag, and it simply folds away.
- You can quickly extend the legs and fold it out, and it's stable just about anywhere.
- Comfortable to sit on for a well-deserved rest on your daily outdoor walk, when gardening, fishing or when waiting in that never-ending queue.
- The Comfort Folding Stool helps to protect your back, knees and joints.
- The Comfort Folding Stool is available in different heights with different seat sizes; 55XL, 65XXL and 75XXL.





- Unfold the stool.
- 2 Pull out the legs and ensure that the red lock buttons are in the correct position and that they are visible after extending the telescopic legs.
- 3 The Comfort Folding Stool is now ready to be used at its normal seat height.
- 4 To fold the stool after use, press the red button and push the telescopic leg back into its non-extended position and fold the stool together.



**5** The Comfort Folding Stool offers two seat heights. When for example gardening at low heights, sit on the Comfort Folding Stool without extending the telescopic legs.



6 To lock the lower legs when the Comfort Folding Stool is folded together, twist the legs so they are locked by the rubber feet being close together. To use the Comfort Folding Stool, do the opposite and you can unlock the lower legs.



1 It can easily be folded together to fit in its carry bag. Ready for the next rest and adventure.

## If You have the "Steady" stability straps for the Comfort Folding Stool

